

# DIPFIT®

**ROTAVIRUS - BIO K 384**  
(BOX OF 10 TESTS)

**TEST STRIPS**  
**FOR DETECTION**  
**OF ROTAVIRUS**  
**IN FAECES**



## HOW TO PROCEED



■ Take the faeces directly from the rectum. If the samples are liquid, take a spoonful.



■ If the samples are solid, remove the excess amount with a spatula or clean object.



■ Dilute the sample with the liquid contained in the dilution tube. Homogenize well, taking care to prevent foam formation.



4


■ Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.



5

■ Plunge a device into the liquid.

■ Wait at most 10 minutes and interpret the result by comparing it with Photo 6.

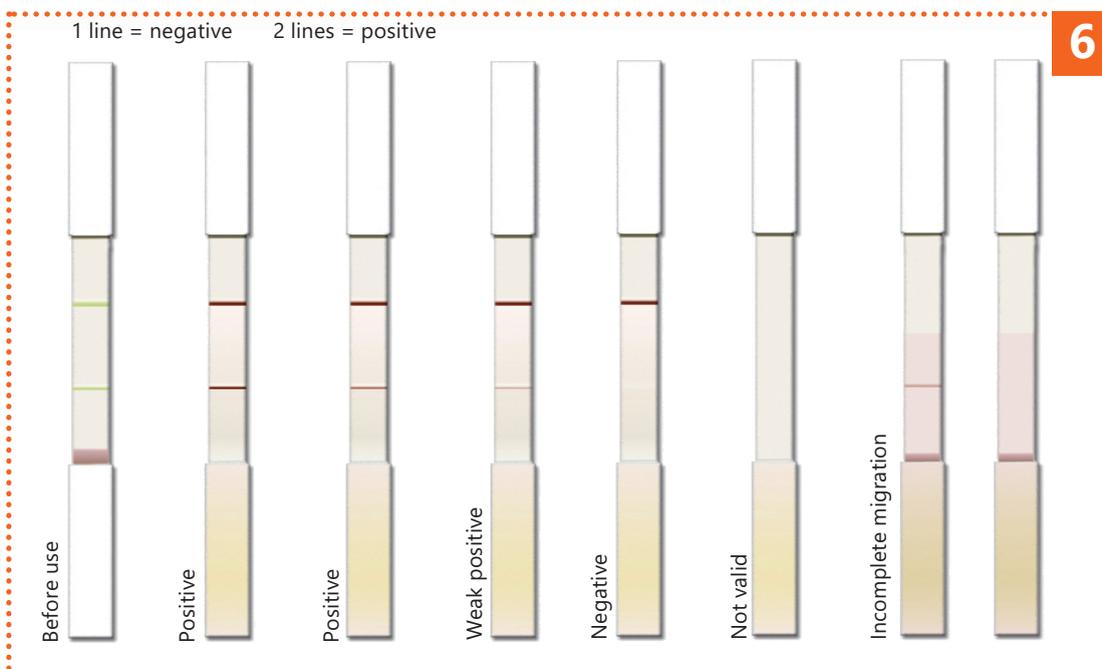

### Precautions for use

- Store kits in a dry place at room temperature.
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.



Bio-X Diagnostics is ISO 9001:2008 certified to assure the best to its customers

**BIO-X DIAGNOSTICS S.A.**

38, rue de la Caléstienne  
5580 Rochefort • BELGIUM  
T. +32(0)84 32 23 77 • F. +32(0)84 31 52 63  
info@biox.com • www.biox.com