

DIPFIT®

ROTAVIRUS - BIO K 373 (BOX OF 10 TESTS)

TEST STRIPS FOR
DETECTION OF PORCINE
ROTAVIRUS IN FAECES



HOW TO PROCEED

1



Take the faeces directly from the rectum. If the samples are liquid, take a spoonful.

2



If the samples are solid, remove the excess amount with a spatula or clean object.

3



Dilute the sample with the liquid contained in the dilution tube. Homogenize well, taking care to prevent foam formation.

DIPFIT®



4


■ Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.



5

■ Plunge a device into the liquid.

■ Wait at most 10 minutes and interpret the result by comparing it with Photo 6.

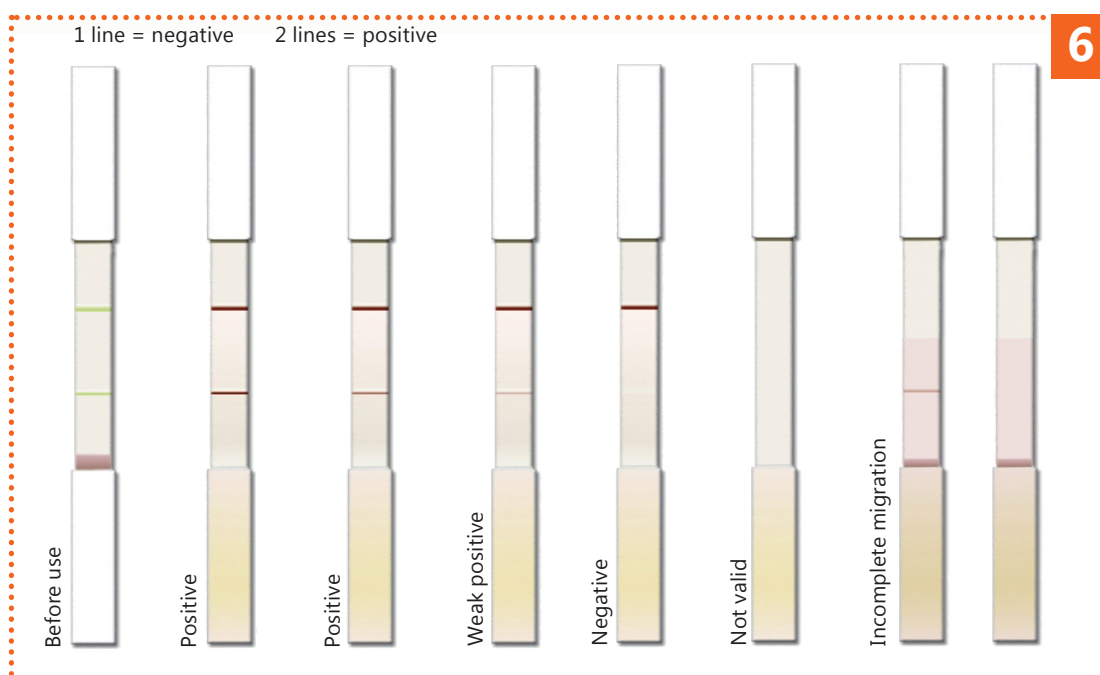

6

Precautions for use

- Store kits in a dry place at room temperature.
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.



Bio-X Diagnostics is ISO 9001:2008
certified to assure the best to its customers

BIO-X DIAGNOSTICS S.A.

38, rue de la Calestienne
5580 Rochefort • BELGIUM
T. +32(0)84 32 23 77 • F. +32(0)84 31 52 63
info@biox.com • www.biox.com