

# DIPFIT<sup>®</sup>

## CRYPTOSPORIDIUM - BIO K 387 (BOX OF 10 TESTS)

**TEST STRIPS FOR  
DETECTION OF  
CRYPTOSPORIDIUM  
IN FAECES**



## HOW TO PROCEED

1



■ Take the faeces directly from the rectum. If the samples are liquid, take a spoonful.

2



■ If the samples are solid, remove the excess amount with a spatula or clean object.

3



■ Dilute the sample with the liquid contained in the bottle. Homogenize well, taking care to prevent foam formation.



4


■ Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.



5

■ Plunge a device into the liquid.

■ Wait at most 10 minutes and interpret the result by comparing it with Photo 6.

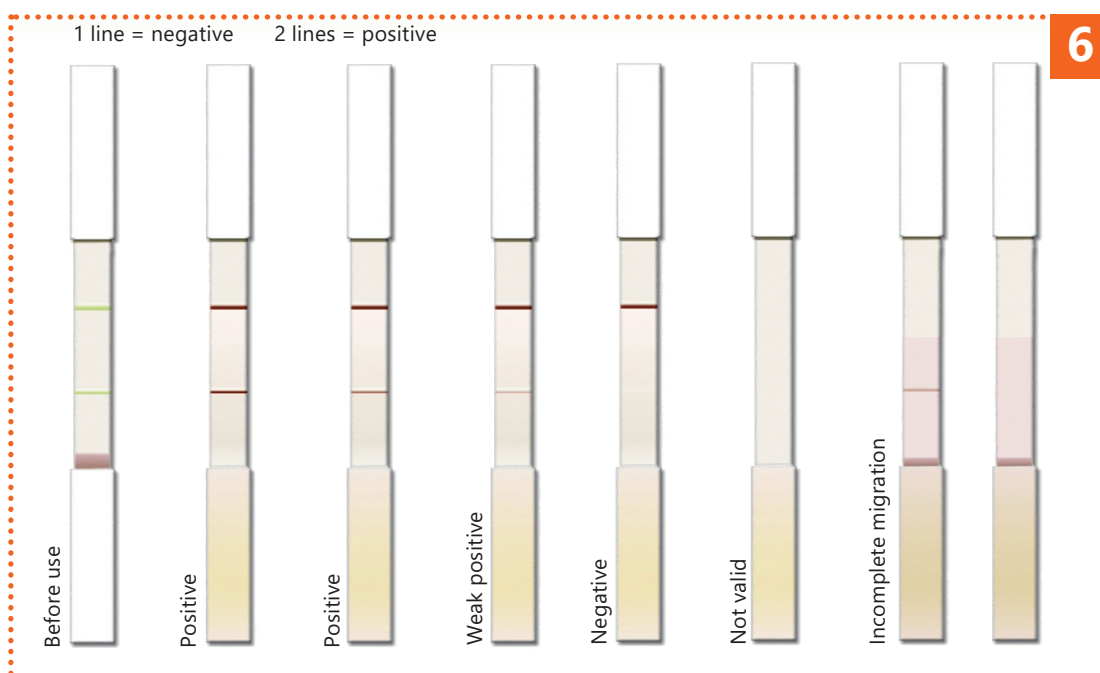

6

### Precautions for use

- Store kits in a dry place at room temperature.
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.