

Adia^X Prep

Device for the preparation of faeces samples

- The easiest sample preparation for ADIAVET™ PARATB REAL TIME
- One device
- Time saving (4X faster)
- 6 elements replaced by 1 element

New procedure for easier sample preparation



Fastest sample preparation on the market:

- Only 1 spoon
- Buffer included (30 mL)



The most precise sample preparation on the market:


- High quantity of faeces (4,5 g)



The most practical sample preparation on the market:

- High filter preventing clogging
- Screw cap preventing splashing when closing
- Screw cap opens in 2 turns with one hand
- Double integral cap
- Standard diameter 50 mL tube: easy transport for 24 series, held on standard rack
- Tightness during transport

Easier sample prep : pre-processing of 96 samples in 1h with ADIAPREP™

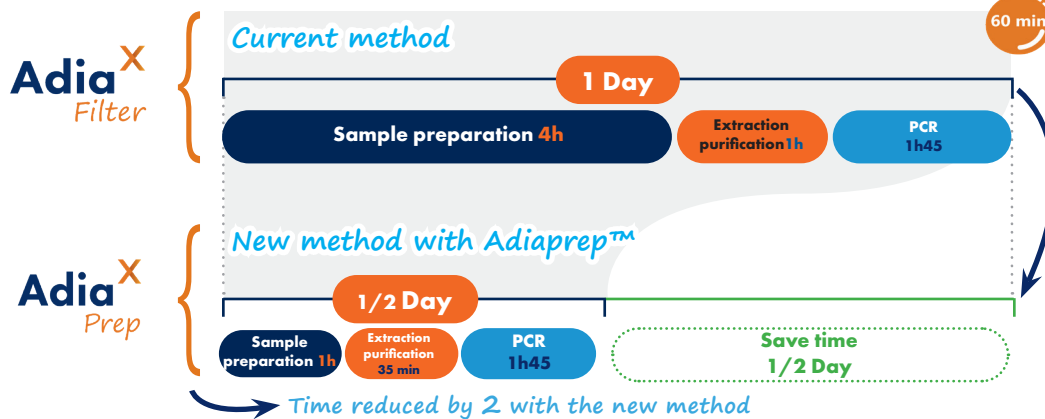

1 element

All-in-one sample preparation device including:


- ✓ No weighing
- ✓ No pipetting
- ✓ No clogging

Easy to use for large amount of samples analysis in a context of herd monitoring

1 Sampling




2 Mix

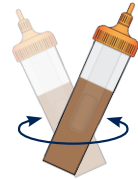

3 Dispense



Sample preparation with ADIAPREP™

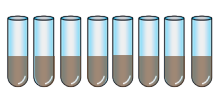

5 min 3000 g



resuspend pellet



5 min - 30 Hz



5 min 3000 g



21 min



PCR

Contact us

✉ support.pcr@biox.com

☎ +33 02 96 68 40 20

🌐 www.biox.com